

## CONTROL PLASMA NORMAL



6 x 1 ml

88 01 97

Axiom normal plasma (as well as high pathological and low pathological plasma) can be used as reference or control for all haemostasis tests requiring citrated plasma without platelets, mainly PT, activated Cephalin, etc. It is of human origin: the absence of Hbs antigen and anti-HIV antibody has been checked; it is however necessary to handle it with the same precautions as plasma of patient. It cannot be used for therapeutic indications.

Axiom normal plasma is made from a great number of donors. It is citrated in PT at 100%  $\pm$  2%. By dilution in an Owren-Koller buffer, it allows the calibration of a thromboplastin.

		Target	Confidence Interval
PT	%	<b>100</b>	---
INR		<b>1</b>	---
Cephalin time	s	<b>33.5</b>	30.5-36.5
Fibrinogen	g/l	<b>2.90</b>	2.70-3.10

### STORAGE

LYOPHILIZED :

see above

RECONSTITUTED :

3 hours after reconstitution with distilled water

### ADDITIONAL PRODUCTS

Axiom HP Plasma	(high pathological)
Axiom LP Plasma	(low pathological)